

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>1</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

<b>Model Name</b>	<b>InnoSight</b>
<b>DICOM Conformance Statement</b>	

Copyright (C) Qisda Healthcare Solution Product Division

The information contained herein is confidential and the property of Qisda Healthcare Solution Product Division and is supplied without liability for errors or omissions. No part may be reproduced or used except as authorized by contract or other written permission. The copyright and the foregoing restriction on reproduction and use extend to all media in which the information may be embodied. Offenders are liable to the payment of damages. All rights reserved in the event of the grant of a patent or the registration of a utility model or design.

Version	Date	Department	Description
0	2017/2/17	HR13	First Release
1	2017/2/17	HR13	Modify worklist query condition
2	2017/10/20	HR13	Modify Model Name to InnoSight
3	2018/07/27	HD12	Add Cine and US Region Calibration Attributes tables(Table 31, 32 in Chapter 5), and add Frame Increment Pointer in Multi-frame Module Attributes in Chapter 5 Table 28. (available in software version 1.0.3 and greater)
4	2018/11/30	HD12	Update DICOM tags. Add Laterality, Patient orientation, Number of stages, Number of views in stage in Chapter 5 Table 24,26,29(available in software version 1.1 and greater)
5	2019/02/14	HD12	Update Pixel Spacing tag value in Chapter 5 Table 27 (available in software version 1.1 and greater)

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>2</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

6	2019/02/22	HD13	Remove SOP class definition and presentation context of Modality Performed Procedure Step in section 3.4.2
6	2019/03/06	HD13	Add chapter 6. Detailed modifications between revisions Modify description of tag “Referring Physician's Name” in Chapter 5 Table 22.
7	2019/03/22	HD13	Add ch. 3.4.2.2.5.1 Setting Current SCP Data Encoding (available in software version 1.1 and greater) Add “Specific Character Set” in Chapter 5 Table 30.
8	2019/12/26	HD13	Remove the SCP Data Encoding that system not support.( Available in software version 1.0.4 and 1.1.1)
8	2020/01/06	HD13	Add “SeriesTime” and “SeriesDate” tag in Chapter 5 Table 24 General Series Module Attributes.(available in software version 1.0.4 and 1.1.1)
9	2020/02/17	HD13	Describe 1.0.4 traceability as followings. 1. the Specific Character Set” in Chapter 5 Table 30 is available in software version 1.0.4 the “SeriesTime” and “SeriesDate” tag in Chapter 5 Table 24 General Series Module Attributes are available in software version 1.0.4.
10	2020/03/12	HD13	Add a tag “InstitutionName” in Chapter 5 Table 25 General Equipment Module Attributes.(available in software version 1.0.4 and 1.1.1)
10	2020/03/23	HD13	Add RLE transfer syntaxes for export function on Table 4 Store - Presentation Context Table and Table 16 SOP classes and transfer syntaxes. (available in software version 1.0.4 and 1.1.1)
10	2020/04/16	HD13	Add additional tag in each table. List as follows. Table 21 Patient Module Attributes: (0010,4000) Patient Comments (0010,2201) Patient Species Description (0010,2292) Patient Breed Description (0010,2297) Responsible Person (0010, 2298) Responsible Person Role

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>3</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

			<p>Table 22 General Study Module Attributes:                  (0040,0244) Performed Procedure Step Start Date                  (0040,0245) Performed Procedure Step Start Time                  (0040,0253) Performed Procedure Step ID                  (0040,0254) Performed Procedure Step Description</p> <p>Table 23 Patient Study Module Attributes:                  (0010,1010) Patient Age                  (0010, 2203) Patient's Sex Neutered</p> <p>Table 24 General Series Module Attributes:                  (0008,0068) Presentation Intent Type</p> <p>Table 25 General Equipment Module Attributes:                  (0008,1090) Manufacturer's Model Name                  (0018,1020) Software Versions</p> <p>Table 26 General Image Module Attributes:                  (0008,0022) Acquisition Date                  (0008,002A) Acquisition DateTime                  (0020,4000) Image Comments</p> <p>Table 29 US Image Module Attributes:                  (0018,6031) Transducer Type                  (0028,0014) Ultrasound Color Data Present                  (0028,2110) Lossy Image Compression                  (available in software version 1.0.4 and 1.1.1)</p>
10	2020/04/30	HD13	Modify tag for Sequence of Ultrasound Regions attribute in Chapter 5 Table 31 US Region Calibration Attributes

□ Product System (PS)

Subject:	DICOM Conformance Statement	Part No.:	9J.3AHUS.000	Rev.:	10
		Doc. No.	9J.3AHUS.000-SD2009-001		
		Component Part. No.:		Page	4 of 24
Model Name:	INNOSIGHT	Marketing Description :			

**1. Definitions**

AE	DICOM Application Entity
DICOM	Digital Imaging and Communications in Medicine
SCU	Service Class User, client
SCP	Service Class Provider, server
SOP	Service Operation Pair
UID	Unique Identifier
VR	Value Representation
FSC	File-Set Creator
FSU	File-Set Updater
MWL	Modality Worklist
US	Ultrasound

**2. Implementation model**

The system implements DICOM services to download worklist from MWL server and export exams with acquired images and cines to a DICOM storage server or USB device.

**3. Networking**

**3.1 Application Data Flow Diagram**

The local storage AE sends exams with acquired images and cines to one remote AE (Storage server). The Ultrasound system starts to send or export exams when “End Exam” is pressed or “Export exam to DICOM” is selected by user.

The worklist AE receives worklist information from remote AE (Worklist server). The Ultrasound System queries a remote AE for worklist items that matches the query request when “searching icon” is pressed by user.

□ Product System (PS)

Subject:	DICOM Conformance Statement	Part No.:	9J.3AHUS.000	Rev.:	10
		Doc. No.	9J.3AHUS.000-SD2009-001		
		Component Part. No.:			Page 5 of 24
Model Name:	INNOSIGHT	Marketing Description :			

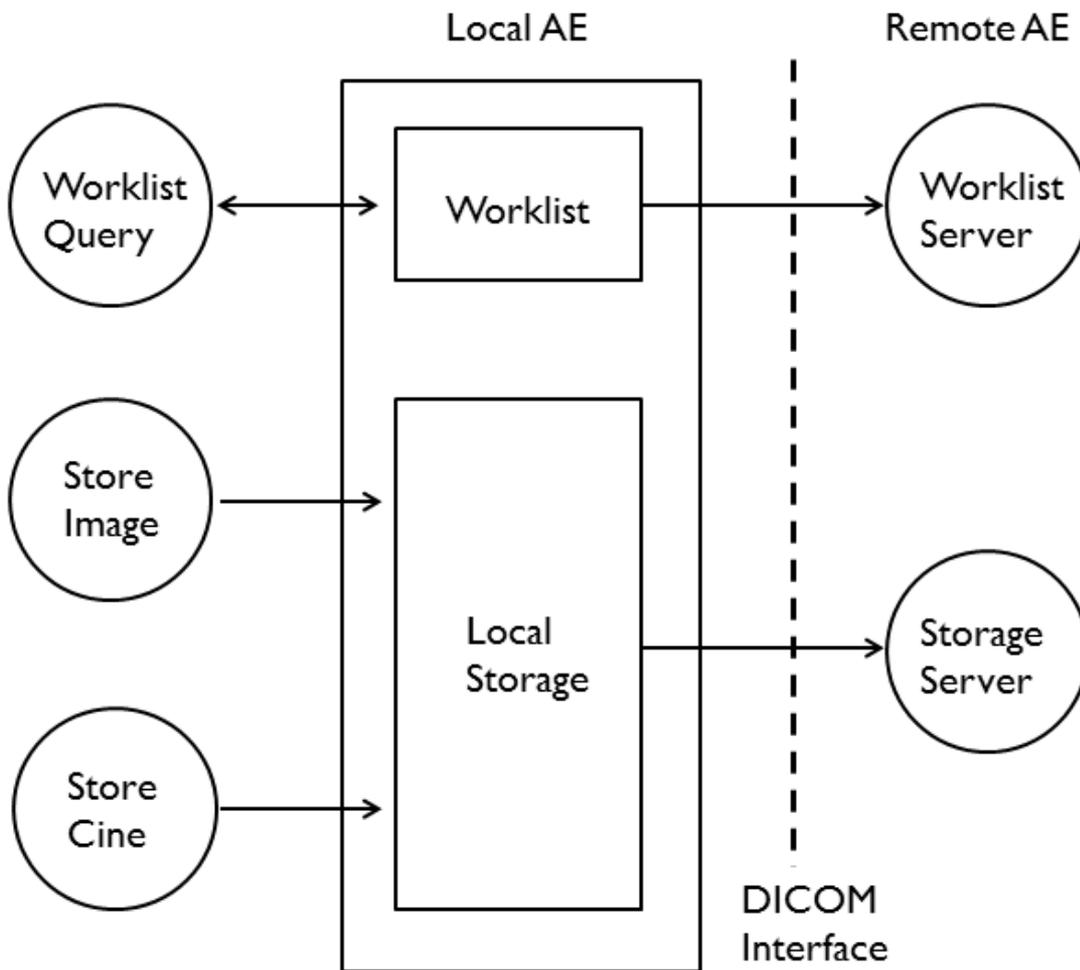


Figure 1 Application data flow diagram

### 3.2 Functional Definitions of AEs

The system has a single AE that performs all the required DICOM related tasks.

- The local storage AE initiates DICOM associations to send images to storage devices using the Ultrasound Image Storage SOP Class. The AE manages image transfer failures. A failed transfer will be re-attempted when network is re-connected.
- The worklist AE receives items from worklist server with studies matching the search criteria by sending C-FIND request. The search criteria contain Name, ID, Accession# and Start Date. The worklist server will transfer all matching items to the Ultrasound system, and the received items will be displayed in the “Query Worklist” page.

### 3.3 Sequencing of Real-World Activities

- Send exam to storage server

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 6 of 24
Model Name: INNOSIGHT	Marketing Description :	

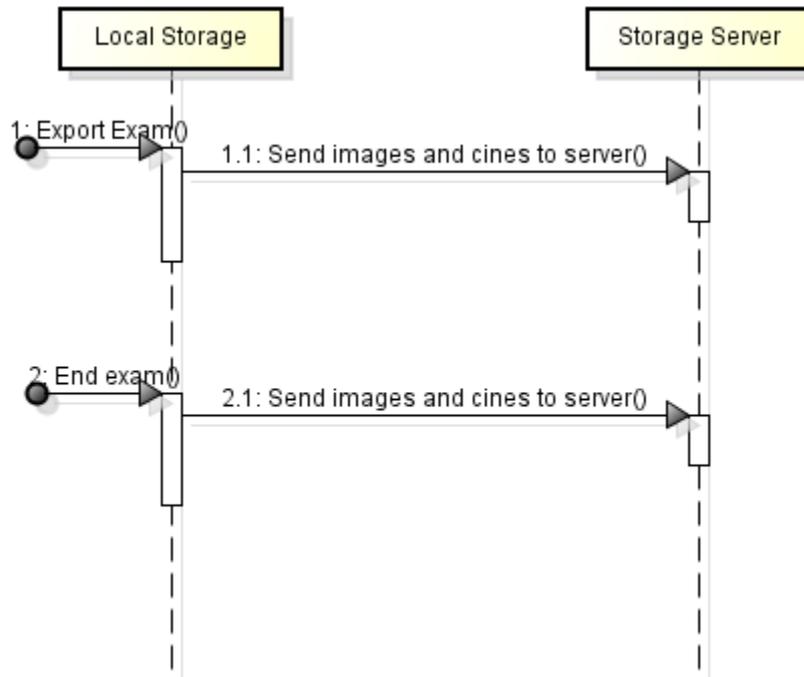


Figure 2 Send exam to storage server

● Query worklist

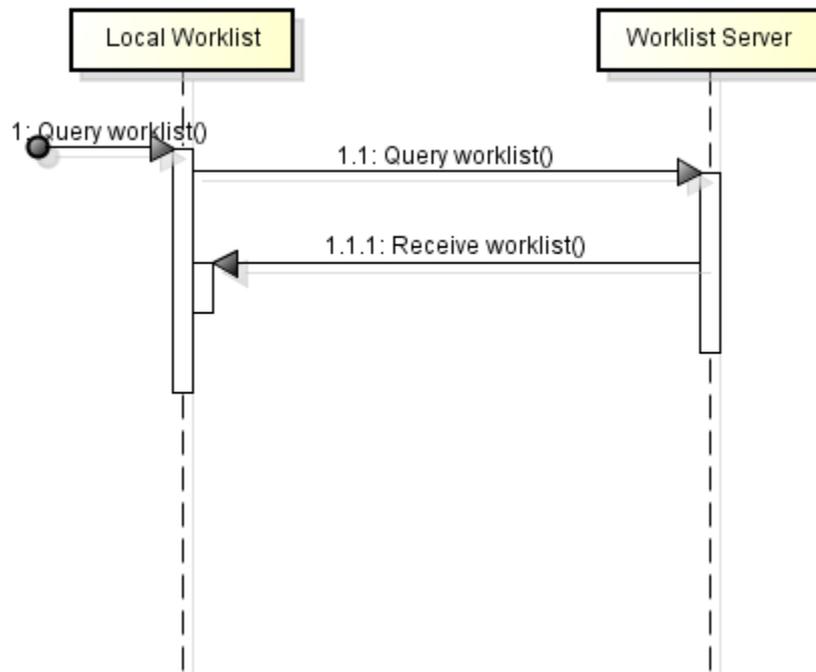


Figure 3 Query worklist

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 7 of 24
Model Name: INNOSIGHT	Marketing Description :	

**3.4 AE Specifications**

**3.4.1 Storage Application Entity Specification**

**3.4.1.1 SOP classes**

This Application Entity provides Standard Conformance to the following DICOM SOP Classes as SCU:

**Table 1 Supported DICOM SOP Classes**

SOP Class Name	SOP Class UID
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1

**3.4.1.2 Association Establishment Policies**

**3.4.1.2.1 General**

The DICOM standard application context name for DICOM 3.0 is proposed:

**Table 2 DICOM Application Context Name**

DICOM Application Context Name	1.2.840.10008.3.1.1.1
--------------------------------	-----------------------

The maximum length PDU for an association initiated by the system is configurable by a company authorized service representative. The default value is 64k (64234) bytes. The Maximum Length PDU negotiation is included in all association establishment requests.

**3.4.1.2.2 Number of Associations**

The AE will only open one association at a time.

**3.4.1.2.3 Asynchronous Nature**

The AE does not support asynchronous mode. All associations use the default synchronous mode for operations.

**3.4.1.2.4 Implementation Identifying Information**

**Table 3 Implementation Identifying Information**

Implementation Class UID	1.2.826.0.1.3680043.9.6741.5.2
Implementation Version Name	MergeCOM3_5_2_0

The Version Name indicated above is subject to change.

**3.4.1.2.5 Association Initiation by Real-World Activity**

The AE will attempt to establish a new association with a remote storage device due to three Real-World Activities.

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 8 of 24
Model Name: INNOSIGHT	Marketing Description :	

- Export exam to DICOM
- End exam
- Failed transfer

All these Real-World Activities result in a **Store** activity, which will generate an identical association initiation and image transfer process. An association is established when one of the actions shown below occurs.

- The user selects DICOM Store (“Export exam to DICOM”) from image review. An association is opened, all images/cines for the exam are transferred and then the association is closed.
- The user selects DICOM Store (“Export exam to DICOM”) from Exam List. An association is opened, all images/cines for the exam are transferred and then the association is closed.
- The user presses “End Exam”. An association is opened, all images/cines for the exam are transferred and then the association is closed.
- The network connection is re-established or a network error condition is cleared. Any images/cines that did not get transferred while there was no network connection will be transferred again. An association is opened, the image(s)/cine(s) transferred, and then the association is closed for the exam.

**3.4.1.2.6 Presentation Context**

The AE will negotiate associations in the order shown in Table 4. The first acceptable syntax will be used.

**Table 4 Store - Presentation Context Table**

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR	1.2.840.10008.1.2.1	SCU	None
		Little Endian			
	Implicit VR	1.2.840.10008.1.2			
	Little Endian				
	Explicit VR	1.2.840.10008.1.2.2			
	Big Endian				
	Run Length	1.2.840.10008.1.2.5			
	Encoding				

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 9 of 24
Model Name: INNOSIGHT	Marketing Description :	

Ultrasound	1.2.840.10008.	Explicit VR	1.2.840.10008.1.	SCU	None
Multi-frame	5.1.4.1.1.3.1	Little	2.1		
Image		Endian			
Storage		Implicit VR	1.2.840.10008.1.		
		Little	2		
		Endian			
		Explicit VR	1.2.840.10008.1.		
		Big Endian	2.2		
		Run			
		Length	1.2.840.10008.1.		
		Encoding	2.5		

**3.4.2 Worklist Application Entity Specification**

**3.4.2.1 SOP classes**

This Application Entity provides Standard Conformance to the following DICOM SOP Classes as SCU:

**Table 5 Supported DICOM SOP Classes**

SOP Class Name	SOP Class UID
MWL Information Model - FIND	1.2.840.10008.5.1.4.31

**3.4.2.2 Association Establishment Policies**

**3.4.2.2.1 General**

The DICOM standard application context name for DICOM 3.0 is proposed:

**Table 6 DICOM Application Context Name**

DICOM Application Context Name	1.2.840.10008.3.1.1.1
--------------------------------	-----------------------

The maximum length PDU for an association initiated by the system is configurable by a company authorized service representative. The default value is 64k (64234) bytes. The Maximum Length PDU negotiation is included in all association establishment requests.

**3.4.2.2.2 Number of Associations**

The AE will only open one association at a time.

**3.4.2.2.3 Asynchronous Nature**

□ Product System (PS)

Subject:	DICOM Conformance Statement	Part No.:	9J.3AHUS.000	Rev.:	10
		Doc. No.	9J.3AHUS.000-SD2009-001		
		Component Part. No.:		Page	10 of 24
Model Name:	INNOSIGHT	Marketing Description :			

The AE does not support asynchronous mode. All associations use the default synchronous mode for operations.

### 3.4.2.2.4 Implementation Identifying Information

**Table 7 Implementation Identifying Information**

Implementation Class UID	1.2.826.0.1.3680043.9.6741.5.2
Implementation Version Name	MergeCOM3_5_2_0

The Version Name indicated above is subject to change.

### 3.4.2.2.5 Association Initiation by Real-World Activity

The AE will attempt to establish a new association with a remote worklist device due to the Real-World Activities.

- Query worklist

The Real-World Activity results in a **Query** activity, which will generate an identical association initiation and worklist query/transfer process. An association is established when the action occurs.

- The user presses "searching icon" in the "Query Worklist" page. An association is opened and the search criterion is transferred. The worklist server returns studies matching the search criteria and then the association is closed.

### 3.4.2.2.6 Presentation Context

The AE will negotiate associations in the order shown in Table 8. The first acceptable syntax will be used.

**Table 8 Worklist - Presentation Context Table**

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
MWL Information Model - FIND	1.2.840.10008.	Explicit VR	1.2.840.10008.1.	SCU	None
	5.1.4.31	Little Endian	2.1		
		Implicit VR	1.2.840.10008.1.		
		Little Endian	2		
		Explicit VR	1.2.840.10008.1.		
		Big Endian	2.2		

□ Product System (PS)

Subject:	DICOM Conformance Statement	Part No.:	9J.3AHUS.000	Rev.:	10
		Doc. No.	9J.3AHUS.000-SD2009-001		
		Component Part. No.:		Page	11 of 24
Model Name:	INNOSIGHT	Marketing Description :			

**3.4.2.3 Worklist Matching Keys**

**Table 9 Worklist – Matching Keys**

Attribute Name	Tag	Type
Patient's Name	(0010,0010)	2
Patient ID	(0010,0020)	2
Accession Number	(0008,0050)	2
Date	(0040,0002)	2C

**3.4.3 Verification Application Entity specification**

**3.4.3.1 SOP classes**

This Application Entity provides Standard Conformance to the following DICOM SOP Classes as SCU:

**Table 10 Supported DICOM SOP Classes**

SOP Class Name	SOP Class UID
Verification	1.2.840.10008.1.1

**3.4.3.2 Association Establishment Policies**

**3.4.3.2.1 General**

The DICOM standard application context name for DICOM 3.0 is proposed:

**Table 11 DICOM Application Context Name**

DICOM Application Context Name	1.2.840.10008.3.1.1.1
--------------------------------	-----------------------

The maximum length PDU for an association initiated by the system is configurable by a company authorized service representative. The default value is 64k (64234) bytes. The Maximum Length PDU negotiation is included in all association establishment requests.

**3.4.3.2.2 Number of Associations**

The AE will only open one association at a time.

**3.4.3.2.3 Asynchronous Nature**

The AE does not support asynchronous mode. All associations use the default synchronous mode for operations.

**3.4.3.2.4 Implementation Identifying Information**

**Table 12 Implementation Identifying Information**

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 12 of 24
Model Name: INNOSIGHT	Marketing Description :	

Implementation Class UID	1.2.826.0.1.3680043.9.6741.5.2
Implementation Version Name	MergeCOM3_5_2_0

The Version Name indicated above is subject to change.

### 3.4.3.2.5 Association Initiation by Real-World Activity

The AE will attempt to establish a new association with a remote storage device due to the Real-World Activities.

- Connection test

The Real-World Activity results in a **Verify** activity, which will generate an identical association initiation and verification process. An association is established when one of the actions shown below occurs.

- The user presses "Test" button in DICOM server setting pages. An association is opened and an echo request is transferred. AE reads message from network and then closes the association (For remote storage and worklist devices).
- The user enters the scan page from settings. An association is opened and an echo request is transferred. AE reads message from network and then closes the association (For remote storage device).
- For a default duration in scan page. An association is opened and an echo request is transferred. AE reads message from network and then closes the association (For remote storage device).

#### 3.4.3.2.5.1 Setting Current SCP Data Encoding

Before query from worklist server, user should set the Current SCP Data Encoding for different customers from different countries. The below table is the support list of SCP Data Encoding.

Character Set Description	Defined Term
Arabic (CP1256)	ISO_IR 127
Arabic (ISO -8859-6)	ISO_IR 127
Central Europe (CP1250)	ISO_IR 101
Central Europe (ISO-8859-2)	ISO_IR 101
Cyrillic (CP1251)	ISO_IR 144
Cyrillic (ISO-8859-5)	ISO_IR 144

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>13</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

English (ASCII)	ISO_IR 6
Greek (CP1253)	ISO_IR 126
Greek (ISO -8859-7)	ISO_IR 126
Hebrew (CP1255)	ISO_IR 138
Hebrew (CP862)	ISO_IR 138
Hebrew (ISO-8859-8)	ISO_IR 138
Japanese (SHIFT_JIS)	ISO_IR 13
Japanese (CP932)	ISO_IR 13
Japanese (ISO-2022-JP)	ISO 2022 IR 87
Japanese (ISO-2022-JP-1)	ISO 2022 IR 87
Japanese (ISO-2022-JP-2)	ISO 2022 IR 87
Korean (CP949)	ISO_IR 149
Korean (JOHAB)	ISO_IR 149
Korean (ISO-2022-KR)	ISO 2202 IR 149
Korean (EUC-KR)	ISO_IR 149
Northern Europe (CP1257)	ISO_IR 110
Northern Europe (ISO-8859-4)	ISO_IR 110
Simplified Chinese (GB18030)	GB18030
Simplified Chinese (GBK)	GB18030
Southern Europe (ISO-8859-3)	ISO_IR 109
Thai (TIS620)	ISO_IR 166
Thai (CP874)	ISO_IR 166
Thai (ISO-8859-11)	ISO_IR 166
Traditional Chinese	ISO_IR 192

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 14 of 24
Model Name: INNOSIGHT	Marketing Description :	

(BIG5)	
Turkish (CP1254)	ISO_IR 148
Turkish (ISO-8859-9)	ISO_IR 148
Vietnamese (CP1258)	ISO_IR 192
Western Europe (CP1252)	ISO_IR 100
Western Europe (CP850)	ISO_IR 100
Western Europe (ISO-8859-1)	ISO_IR 100
Unicode (UTF-8)	ISO_IR 192

**3.4.3.2.6 Presentation Context**

The AE will negotiate associations in the order shown in Table 13. The first acceptable syntax will be used.

**Table 13 Verify - Presentation Context Table**

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Verification	1.2.840.10008.1.1	Explicit VR Little Endian Implicit VR	1.2.840.10008.1.2.1 1.2.840.10008.1.	SCU	None

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 15 of 24
Model Name: INNOSIGHT	Marketing Description :	

	Little Endian	2		
	Explicit VR	1.2.840.10008.1.		
	Big Endian	2.2		

**3.4.3.3 Request Message Table**

**Table 14 Association Negotiation Request message contents for each DICOM device**

Device Type	SOP Classes Requested
Storage SCP	US Image Storage US Multi-frame Storage Verification
MWL SCP	Modality Worklist Verification

**4. Media Storage**

**4.1 Application Data Flow Diagram**

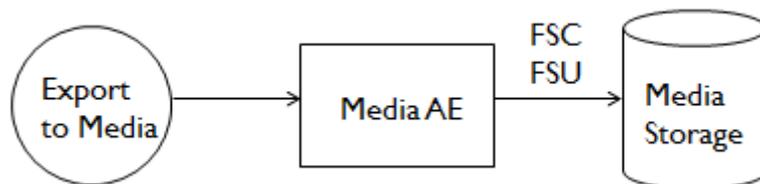


Figure 4 Data flow diagram for media storage

**4.2 Functional Definitions of AE**

The media AE exports the selected exam to the selected media destination.

**4.3 AE Specifications**

**4.3.1 Implementation Identifying Information**

**Table 15 Implementation Identifying Information**

Implementation Class UID	1.2.826.0.1.3680043.9.6741.5.2
Implementation Version Name	MergeCOM3_5_2_0

The Version Name indicated above is subject to change.

**4.3.2 Real-World Activity**

- The user selects DICOM Export (“Export exam (DICOM) to external storage”) from image review. A corresponding DICOMDIR is created and written to the media.

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 16 of 24
Model Name: INNOSIGHT	Marketing Description :	

- The user selects DICOM Export (“Export exam (DICOM) to external storage”) from Exam List. A corresponding DICOMDIR is created and written to the media.

### 4.3.3 Options

**Table 16 SOP classes and transfer syntaxes**

Information Object Definition	SOP Class UID	Transfer Syntax	Transfer Syntax UID
Media Storage Directory Storage	1.2.840.10008.1.3.10	Explicit VR Little Endian Run Length Encoding	1.2.840.10008.1.2.1 1.2.840.10008.1.2.5
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	Explicit VR Little Endian Run Length Encoding	1.2.840.10008.1.2.1 1.2.840.10008.1.2.5
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	Explicit VR Little Endian Run Length Encoding	1.2.840.10008.1.2.1 1.2.840.10008.1.2.5

The (DICOMDIR) Directory includes Directory Records of PATIENT, STUDY, SERIES, IMAGE

**Table 17 Patient Directory Record**

Attribute Name	Tag	Type	Description
Patient's Name	(0010,0010)	2	Patient name in the form

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>17</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

			FIRST MIDDLE LAST based on the value entered in the Name field on the New Patient screen. Always sent even if zero length.
Patient ID	(0010,0020)	2	As entered in the ID field on the New Patient screen. Always sent even if zero length.

**Table 18 Study Directory Record**

Attribute Name	Tag	Type	Description
Study Instance UID	(0020,000D)	1	System generated
Study Date	(0008,0020)	1	Date when study was started
Study Time	(0008,0030)	2	Time when study was started

**Table 19 Series Directory Record**

Attribute Name	Tag	Type	Description
Modality	(0008,0060)	1	Always "US"
Series Instance UID	(0020,000E)	1	System generated
Series Number	(0020,0011)	2	A number unique with the study

**5. SOP Specific Conformance to Ultrasound Image Storage SOP Class**

The AE uses the Ultrasound Image IOD Modules as shown in the tables below. Table 20 shows the modules used and Table 20 through 32 show the specific details of the attributes used in those modules. The only DIMSE service applicable to the combined IOD modules is C-STORE.

**Table 20 Ultrasound Image IOD Modules Used**

Information Entity	Module Name	Reference
Patient	Patient	See Table 21
Study	General Study	See Table 22
	Patient Study	See Table 23
Series	General Study	See Table 24
Equipment	General Equipment	See Table 25
Image	General Image	See Table 26
	Image Pixel	See Table 27

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 18 of 24
Model Name: INNOSIGHT	Marketing Description :	

	Multi-frame Present	See Table 28
	US Image	See Table 29
	SOP Common	See Table 30
	US Region Calibration	See Table 31
	Cine	See Table 32

**Table 21 Patient Module Attributes**

Attribute Name	Tag	Type	Description
Patient's Name	(0010,0010)	2	Patient name in the form FIRST MIDDLE LAST based on the value entered in the Name field on the New Patient screen. Always sent even if zero length.
Patient ID	(0010,0020)	2	As entered in the ID field on the New Patient screen. Always sent even if zero length.
Patient's Birth Date	(0010,0030)	2	As entered in the DOB field on the New Patient screen. Always sent even if zero length.
Patient's Sex	(0010,0040)	2	Set based on the Sex field on the New Patient screen. "M" for Male, "F" for Female.
Patient Comments	(0010,4000)	3	No UI to fill, always empty
Patient Species Description	(0010,2201)	1C	No UI to fill, always empty
Patient Breed Description	(0010,2292)	2C	No UI to fill, always empty
Responsible Person	(0010,2297)	2C	No UI to fill, always empty
Responsible Person Role	(0010,2298)	1C	Required if Responsible Person is present and has a value. Always "OWNER".

**Table 22 General Study Module Attributes**

Attribute Name	Tag	Type	Description
Study Instance UID	(0020,000D)	1	System generated
Study ID	(0020,0010)	2	ID for the Study
Study Date	(0008,0020)	1	Date when study was

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>19</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

			started
Study Time	(0008,0030)	2	Time when study was started
Accession Number	(0008,0050)	2	As entered in the Accession field on the Patient Information screen. Always sent even if zero length.
Referring Physician's Name	(0008,0090)	2	Name of the Referring Physician's
Performed Procedure Step Start Date	(0040,0244)	3	This value may be used to determine the earliest date to use as Study Date
Performed Procedure Step Start Time	(0040,0245)	3	This value may be used to determine the earliest time to use as Study Time
Performed Procedure Step ID	(0040,0253)	3	This value combines with Performed Procedure Step Start Date and Time.
Performed Procedure Step Description	(0040,0254)	3	Institution-generated description or classification of the Procedure Step that was performed

**Table 23 Patient Study Module Attributes**

Attribute Name	Tag	Type	Description
Patient's Size	(0010,1020)	3	
Patient's Weight	(0010,1030)	3	
Patient's Age	(0010,1010)	3	Age of the Patient.
Patient's Sex Neutered	(0010,2203)	2C	Whether or not a procedure has been performed in an effort to render the patient sterile.

**Table 24 General Series Module Attributes**

Attribute Name	Tag	Type	Description	Comment
Modality	(0008,0060)	1	Always "US"	
Protocol Name	(0018,1030)	3	Always "Free From"	
Series Instance UID	(0020,000E)	1	System generated	
Series Number	(0020,0011)	2	A number unique with the study	

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>20</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

Laterality	(0020,0060)	2C	Always "null"
Series Date	(0008,0021)	3	Date the Series started.
Series Time	(0008,0031)	3	Time the Series started.
Presentation Intent Type	(0008,0068)	3	Always " FOR PRESENTATION"

**Table 25 General Equipment Module Attributes**

Attribute Name	Tag	Type	Description
Manufacturer	(0008,0070)	2	Name of Manufacturer
Station Name	(0008,1010)	3	
Institution Name	(0008,0080)	3	Institution where the equipment that produced the Composite Instances is located.
Manufacturer's Model Name	(0008,1090)	3	Manufacturer's model name of the equipment that produced the Composite Instances.
Software Versions	(0018,1020)	3	Manufacturer's designation of software version of the equipment that produced the Composite Instances.

**Table 26 General Image Module Attributes**

Attribute Name	Tag	Type	Description
Image Type	(0008,0008)	3	Always "ORIGINAL\PRIMARY" and application name
Instance Number	(0020,0013)	2	A number that identifies this image.
Content Date	(0008,0023)	2C	System generated
Content Time	(0008,0033)	2C	System generated
Patient Orientation	(0020,0020)	2C	Always "null"
Acquisition Date	(0008,0022)	3	The date the acquisition of data that resulted in this image started
Acquisition Time	(0008,0032)	3	The time the acquisition of data that resulted in this image started
Acquisition DateTime	(0008,002A)	3	The date and time that the acquisition of data that

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 21 of 24
Model Name: INNOSIGHT	Marketing Description :	

			resulted in this image started.
Image Comments	(0020,4000)	3	User-defined comments about the image

**Table 27 Image Pixel Module Attributes**

Attribute Name	Tag	Type	Description
Samples per Pixel	(0028,0002)	1	A value of 3 for "RGB"
Photometric Interpretation	(0028,0004)	1	Always "RGB"
Planar Configuration	(0028,0006)	1C	Always 0000H
Rows	(0028,0010)	1	Always 706
Columns	(0028,0011)	1	Always 960
Pixel Spacing	(0028,0030)	1C	Physical distance between the center of each pixel (Row Spacing \ Column Spacing , unit: mm)
Bits Allocated	(0028,0100)	1	Always 0008H
Bits Stored	(0028,0101)	1	Always 0008H
High Bit	(0028,0102)	1	Always 0007H
Pixel Representation	(0028,0103)	1	Always 0000H
Pixel Data	(7FE0,0010)	1	Data of image pixels

**Table 28 Multi-frame Module Attributes**

Attribute Name	Tag	Type	Description
Number of Frames	(0028,0008)	1	Number of frames in a Multi-frame Image
Frame Increment Pointer	(0028,0009)	1	Always (0018,1063).

**Table 29 US Image Module Attributes**

Attribute Name	Tag	Type	Description
Image Type	(0008,0008)	3	Always "ORIGINAL\PRIMARY" and application name
Number of Stages	(0008,2124)	2C	Always "null"
Number of Views in Stage	(0008,212A)	2C	Always "null"

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>22</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

Samples per Pixel	(0028,0002)	1	A value of 3 for "RGB"
Photometric Interpretation	(0028,0004)	1	Always "RGB"
Planar Configuration	(0028,0006)	1C	Always 0000H
Bits Allocated	(0028,0100)	1	Always 0008H
Bits Stored	(0028,0101)	1	Always 0008H
High Bit	(0028,0102)	1	Always 0007H
Pixel Representation	(0028,0103)	1	Always 0000H
Lossy Image Compression	(0028,2110)	1C	Specifies whether an Image has undergone lossy compression
Ultrasound Color Data Present	(0028,0014)	3	This element indicates if any ultrasound color data is present in an image.
Transducer Type	(0018,6031)	3	Defined Terms: SECTOR_PHASED LINEAR CURVED LINEAR ENDOCAV_CLA

**Table 30 SOP Common Module Attributes**

Attribute Name	Tag	Type	Description
SOP Class UID	(0008,0016)	1	1.2.840.10008.5.1.4.1.1.6.1 or 1.2.840.10008.5.1.4.1.1.3.1
SOP Instance UID	(0008,0018)	1	System generated
Specific Character Set	(0008,0005)	1C	Character Set that expands or replaces the Basic Graphic Set

**Table 31 US Region Calibration Attributes**

Attribute Name	Tag	Type	Description
Sequence of Ultrasound Regions	(0018,6011)	1	Always "null"
Physical Units X Direction	(0018,6024)	1	The physical units of the

□ Product System (PS)

Subject: <b>DICOM Conformance Statement</b>	Part No.: <b>9J.3AHUS.000</b>	Rev.: <b>10</b>
	Doc. No. <b>9J.3AHUS.000-SD2009-001</b>	
	Component Part. No.:	Page <b>23</b> of <b>24</b>
Model Name: <b>INNOSIGHT</b>	Marketing Description :	

			dimensions of the region.
Physical Units Y Direction	(0018,6026)	1	The physical units of the dimensions of the region.
Region Location Min X0	(0018,6018)	1	The bounds of a rectangle specifying the location of the region x0.
Region Location Min Y0	(0018,601A)	1	The bounds of a rectangle specifying the location of the region y0.
Region Location Min X1	(0018,601C)	1	The bounds of a rectangle specifying the location of the region x1.
Region Location Min Y1	(0018,601E)	1	The bounds of a rectangle specifying the location of the region y1.
Physical Delta X	(0018,602C)	1	The physical value increments per positive X pixel increment.
Physical Delta Y	(0018,602E)	1	The physical value increments per positive Y pixel increment.
Region Spatial Format	(0018,6012)	1	The spatial organization of the data within the region. Always "1"
Region Data Type	(0018,6014)	1	The type of data within the region. Tissue is 1, Color Flow is 2, PW Spectral Doppler is 3, CW Spectral Doppler is 4, and Area Trace is 10.
Region Flags	(0018,6016)	1	Flags used for special handling of the region.

□ Product System (PS)

Subject: DICOM Conformance Statement	Part No.: 9J.3AHUS.000	Rev.: 10
	Doc. No. 9J.3AHUS.000-SD2009-001	
	Component Part. No.:	Page 24 of 24
Model Name: INNOSIGHT	Marketing Description :	

**Table 32 Cine Attributes**

Attribute Name	Tag	Type	Description
Frame Time	(0018,1063)	1C	Nominal time (in msec) per individual frame. Always 50.
Cine Rate	(0018,0040)	3	Number of frames per second. Always 20.

**6. Detailed modifications between revisions**

The detailed modifications are listed here

- Add extra information " Row Spacing \ Column Spacing" in description of Pixel space tag (Chapter 5 Table 27).
- The description of tag "Referring Physician's Name" was "Name of the patient's" and is modified to "Name of the Referring Physician" (Chapter 5 Table 22).